# FIRST AID KIT

## FOR CARS AND VANS

### KIT PRONTO SOCCORSO

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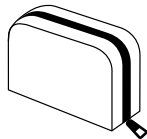
### KIT DE PREMIERS SECOURS

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
### VERBANDKASTEN

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
- ① MANUALE DI PRONTO SOCCORSO
- ② FIRST AID MANUAL
- ③ MANUEL DE PREMIÈRES SECOURS
- ④ ERSTE HILFE BROSCHÜRE

Art. 66962

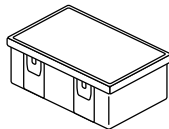Art. 66963

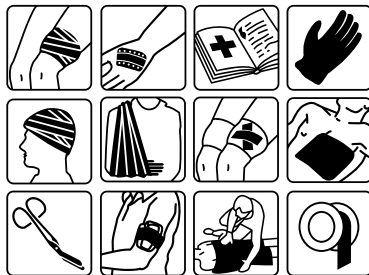**I**

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**GB**

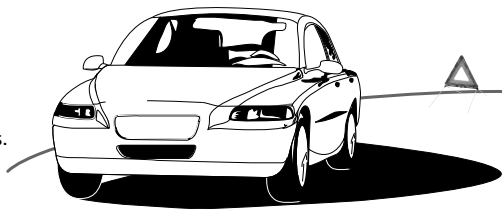
Allow ambulances to pass between the waiting vehicles.  
First of all safeguard the site of the accident,  
put up the warning triangle.  
Assess the situation cautiously and considerately.

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**GB**

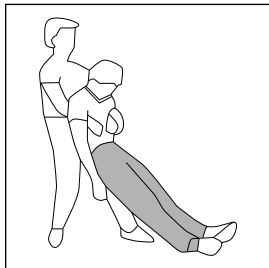
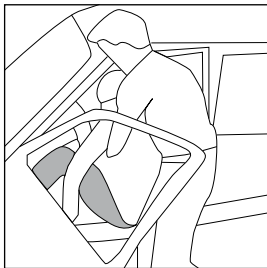
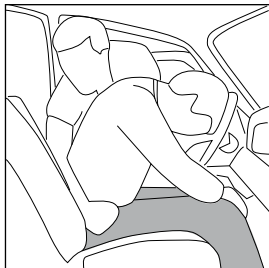
Address casualty, open the safety belt and grasping his/her hips, turn casualty's back towards you.  
Get a hold of casualty's forearm passing your arms under his armpits. Drag casualty out of the vehicle and put down in a safe place.

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GB

Open the visor and address casualty.

**If not responsive:** remove crash helmet by two first aiders as shown below. Check breathing.

**If casualty is breathing:** recovery position.

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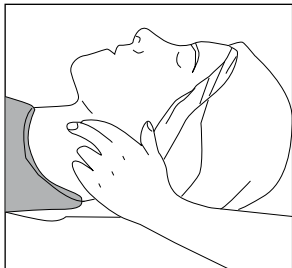
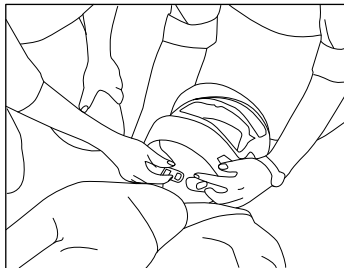
Patient does not respond.

**Respiratory failure:**

No breath to be heard, no movement of the chest.

**Circulatory failure:**

No signs of life like breathing, movement, swallowing, coughing.

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**GB**

If not responsive, check breathing:  
if breathing is present **Recovery position.**  
Stretch near side arm and push it  
under the hip. Bend near side leg.  
(fig. 1 / 2)

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**GB**

Get ahold of far side shoulder and hip and turn casualty cautiously on his side. (fig. 4)

Recline his/her head, open his/her mouth, and bend backward arm. (fig. 5-6)

Keep casualty warm and call for the ambulance.

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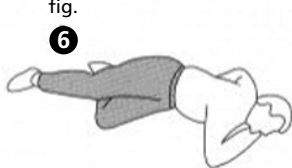
**GB**

In case of respiratory failure -> artificial respiration

1. Recline head.
  2. Keep mouth or nose closed.
  3. Give breath into mouth or nose.
- Turn your head and watch casualty's expiration.  
Give 15 breaths per minute.

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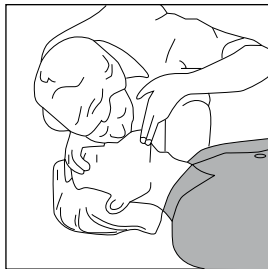
GB

Circulatory failure -> Cardio-pulmonary Resuscitation. Continue CPR until a doctor is available, applying artificial breathing and chest compression (100/min) at a rate of 2:15.

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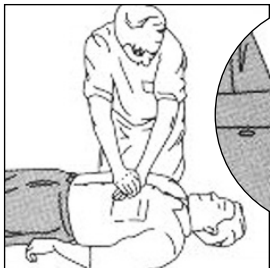
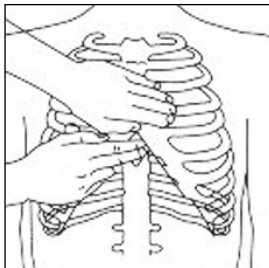
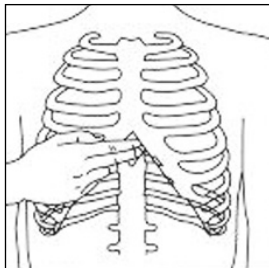
**GB**

In case of severe bleedings from the arm, first elevate arm and apply indirect pressure at the upper arm. Dress severe bleedings from arm or leg by applying a pressure bandage as shown here.

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GB

Casualties who have a shock are pale, they are cold and they have a feeble pulse.

Place casualty with his legs in an elevated position, reassure him and keep him warm.

There should be a rescue foil in the first aid kit.

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**GB**

In case of injuries or illnesses in the abdominal region, place casualty with bent knees for relaxation as shown here. In case of breathing difficulties or cardiac infarct, place casualty with head and chest elevated if he is responsive.

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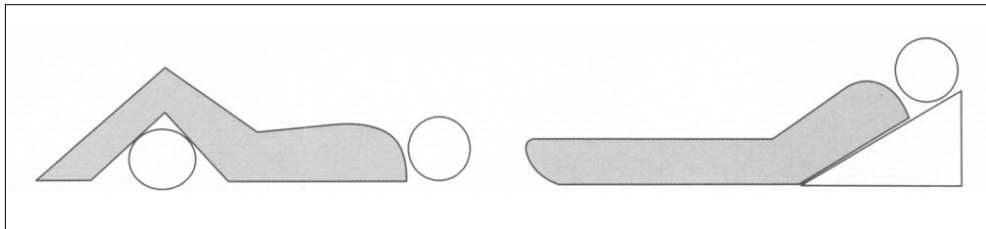
GB

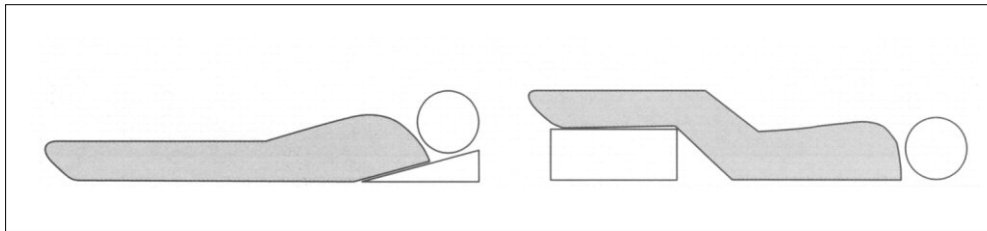
In case of head injuries such as a concussion of the brain: place casualty with his head elevated if he is responsive.  
A casualty who has a shock is positioned on his back with his legs elevated if he is responsive.

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- GB** Sterilized part remain  
sterile as long as the packing  
is not damaged

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**GB**

Ambient conditions may impair the usability of some products, please check regularly. Please renew products after the use-by date.

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## ⓐ SPECIFICATION CONTENTS

### Contents according to DIN 13164 - 2022

- 1 Adhesive tape, 5m x 2,5 cm €€
- 14 plasters (assorted sizes) €€
- \* 1 First aid dressing bandage 10x12cm €€ 2797
- \* 2 First aid dressing bandage 8x10cm €€ 2797
- \* 1 First aid dressing bandage 10x12cm €€ 2797
- \* 1 Absorbent non-woven compress 60x80cm €€ 2797
- 2 Elastic bandages, 6cm x 4m €€
- 3 Elastic bandages, 8cm x 4m €€
- 1 Emergency blanket €€
- \* 6 Wound dressings, 10cm x 10cm €€ 2797
- 1 Triangular bandages 96x96x136cm €€
- 1 Pair of First Aid Scissors €€
- 4 Disposable gloves €€
- 2 Wet wipes
- 2 Disposable medica mask €€
- 1 First Aid Instruction booklet
- 1 List of content

\* Products for the treatment of wounds should be replaced after the expiry date.

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**LAMPA S.p.A.**